




R&B Wire Products, Inc.


ASSEMBLY INSTRUCTIONS

PST – TELESCOPING PRIVACY SCREEN


TOOLS REQUIRED: Phillips screwdriver, Electric drill, 1/8" drill bit and a tape measure.

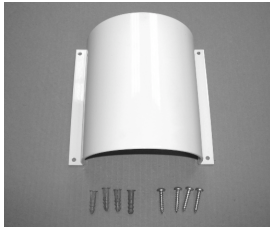

- 1) The recommended mounting height of the bracket is 80" from the floor to the top of the bracket. To mount the Bracket, locate a wall stud and align one side of the bracket along the stud. Mark the 4 holes in the bracket. Using the 1/8" drill bit, drill a hole at the 4 places marked on the wall. Insert 2 of the Wall anchors into the holes that do not line up with the wall stud. Using the screws supplied, fasten one side of the bracket to the wall and into the stud. Fasten the other side of the bracket into the wall anchors. If a wall stud can not be located, use all four wall anchors and screws supplied. Periodically check mounting screws for tightness.

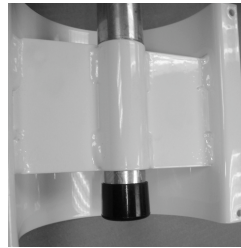

- 2) Insert the short end of the Telescoping Rod into the hole in the bracket after the bracket has been mounted and secured to the wall. Place the Black Plastic Cap on the bottom of the short end. The expandable arm should swing freely from side to side.

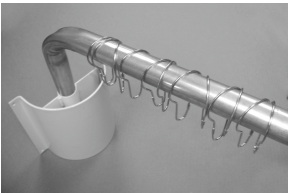

- 3) Place the supplied Curtain Rings onto the rod.



- 4) Attach the Curtain to the hooks. There are 12 hooks provided, only 11 are needed.



- 5) Your Telescoping Privacy Screen is now ready for use. To clean the vinyl panel use only mild liquid soap and water.

WARNING

- Do not hang equipment or personal objects from curtain rod. Do not use the rod as an aid to assist a patient.



To Purchase - www.UNOCLEAN.com

phone: 888-226-2724



"YOUR RELIABLE PARTNER
FOR OVER 57 YEARS"